

## Before Use

1. Check for any damage, foreign substances, or contaminants in the product before use.
2. Do not reuse the mixing tip after it has been used.
3. This device is exclusively for use with 50ml dental heavy body impression material; do not use with other types of impression materials.

## Usage Instructions and Operating Procedures

1. Ensure the plunger is retracted before connecting the impression material.
2. Remove the cap from the cartridge and vertically secure it to the cartridge attachment section.
3. Attach the mixing tip to the impression material.
4. Press the start button to operate the device.
5. When replacing the impression material cartridge:
  - ① Use the reverse button to fully retract the plunger.
  - ② Remove the impression material cartridge by pulling it upwards.

## Cautions

1. Use a power cord that meets the specified standards for C-type charging.
2. Keep the product away from flammable materials to prevent explosions or fires.
3. Protect the product from moisture to prevent electric shock or malfunction.
4. Handle the product carefully to avoid damage from drops.
5. Do not place or use the product in high temperatures or direct sunlight to prevent internal failure, smoke, or fire.
6. Do not touch the end of the charger with fingers or conductive objects to avoid overheating or fire hazards.
7. Do not frequently insert or remove the product or cable with wet hands to avoid the risk of electric shock.
8. If not using the product for an extended period, disconnect the charger and store it separately.
9. This product is intended only for use with dental heavy body impression material. Using it with anything other than heavy body impression material may damage the device.
10. Do not disassemble the product. This can cause malfunction, injury, electric shock, or fire.
11. Overcharging the product can shorten the battery life.

## Customer Service

If the product malfunctions during use, customers can receive free service within six months from the date of purchase. However, if the malfunction is due to external impact, customer negligence, or natural disasters, repairs will be charged even within the warranty period.



# Quick start guide

## Product Usage and Quick Guide



1	Cartridge Insertion Slot	2	Plunger Position Indicator	3	Start Button
4	Charging Port (C-type)	5	Battery Level Indicator (LED)	6	Reverse Button